

Blood Test Soil

Make Clean Seen

- Whole Blood Soil
- Composed of Albumin and Hemoglobin
- Resembles blood on surfaces of patient-used instruments
- Simple: Lyophilized, ready-to-use
- Stable: Longer shelf-life than fresh blood

INSTRUCTIONS FOR USE:

- Hydrate soil in 7 ml* of sterile DI or RO water.

*NOTE: Less water may be used to obtain a thicker soil, yet more time may be needed to dissolve the soil.

- Dissolve by shaking within 15 minutes of rehydrating.
- Coat desired instrument(s).



healthmark
INDUSTRIES CO
health care products

Healthmark

33671 Doreka Drive
Fraser, MI
48206

Phone: 586-774-7600 X 6682
Fax: 586-774-6473
E-mail:
kaumudi@healthmark.info

