

Instructions for Use: Saliva Test Soil

Brand Name of Product	Saliva Test Soil
Generic Name of Product	Saliva Test Soil, Modified Saliva Test Soil
Product Code Number(s)	STS-100ML, STS-25ML, STS-NH-39ML,
Intended Use	Simulated use saliva based test soil for cleaning validations of oral devices.
Range of Applications for Product	For simulated use soiling of items for cleaning validations for oral devices.
Key Specifications of Product	<ul style="list-style-type: none"> • STS-100ML, STS-25ML- Mixture of purified bovine proteins (hemoglobin, albumin), artificial saliva, mucin. • STS-NH-39ML-Mixture of purified bovine albumin, artificial saliva, mucin. No hemoglobin.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	<ul style="list-style-type: none"> • Store the separate vials with dry and wet components at room temperature. • Keep away from light and heat.
Packaging Contents	N/A
Shelf Life	3 months from the date of manufacture

Instructions for Using Product	
Description of Use(s)	N/A
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Add the provided solution S to the dry soil. 2. Dissolve the components by shaking for 5 minutes. A laboratory vortex mixer may be used. Use within approximately 15 minutes of reconstituting. 3. Contaminate the item in a manner that reflects real-life soiling from use.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Hazard Information <ul style="list-style-type: none"> ○ May be harmful if ingested or inhaled. ○ May cause skin irritation. ○ May cause eye irritation. ○ May be irritant to mucous membranes and upper respiratory tract. • First Aid Measures <ul style="list-style-type: none"> ○ After skin contact: Wash off with plenty of water. Remove contaminated clothing. ○ After eye contact: Rinse with plenty of water with the eyelid held wide open. Call an Ophthalmologist. ○ After swallowing: Make the victim drink plenty of water. Call a physician. • Accidental Release Measures <ul style="list-style-type: none"> ○ After spillage: Dilute spilled substance with plenty of water and absorb with absorbent material.
Disposal	Dispose of in accordance with regional and/or national regulations.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A

Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	N/A
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com www.artificialtestsoil.com